bachmaier® comfort

①		
DE	Gebrauchsanweisung	1
EN	Instructions starting at page	2
FR	Le manuel commence à la page	2
IT	Istruzioni a partire da pagina	3
ES	Instrucciones desde la página	4
PT	Instruções da página	4
NL	Instructies vanaf pagina	5
SV	Instruktioner från sidan	6
DA	Instruktioner fra side	6
FI	Ohjeet sivulta	7
NO	Instruksjoner fra side	8
PL	Instrukcje ze strony	8
HU	Utasítások az oldalról	9
HR	Upute sa str	10
TR	Sayfadaki talimatlar	10

bachmaier® comfort:

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bachmaier® comfort:

Intended use: Wide-ranging hearing protection with a special design. The customised bachmaier® comfort hearing protection attenuates dangerous or disturbing ambient noise to a comfortable level without eliminating all sounds - because for safety reasons it is important to hear specific sounds at all times. The individual designs provide high wearing comfort and easy handling, perfectly matched to the personal area of application.

Special areas of application: DIY projects | gardening | snow blowing | everyday noise at home and while traveling | libraries and universities | nightlife and events | noise-induced sleep disturbances.

Instructions for correct use: The customised hearing protection is reusable. It is worn in the ear canal or in the auricle and protects against dangerous or disturbing noise when correctly used and fitted in the ear. Hold the handle or support flap with your thumb and forefinger and push the hearing protection into the ear canal with light pressure until it fits comfortably and completely seals the ear canal.

IMPORTANT: The earmold marked 'R' fits in the right ear, the one marked 'L' in the left ear.

The hearing protection only provides its full protective effect if it fits correctly in the ear and is worn continuously in noisy situations. Have its fit and function checked regularly by a qualified dealer, as changes to the ear canal or damage can have a negative impact on the protective function. In this case, the earmould should be replaced. Ensure that the hearing protection is used correctly and checked in accordance with the manufacturer's instructions. If these recommendations are not followed, the protective effect may be impaired. The product is not subject to a limited shelf life. The lifetime depends on the use (wearing frequency, wearing time) and care of the product and can be up to 5 years.

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Storage and care: Store your hearing protection in the supplied case at a maximum temperature of +55°C. Care for the hearing protection after each use by removing any ear wax residue with the wax stick provided and cleaning the earmolds with water, some mild soap or a damp cloth.

This product can be negatively affected by certain chemical substances. For further information, please contact the manufacturer.

Cleaning and Care with oto-fresh®: Discover our oto-fresh® products for optimal cleaning and care of your bachmaier® hearing protection! Available at your hearing care professional.

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bachmaier GmbH