bachmaier® water

①	
DE	Gebrauchsanweisung
EN	Instructions starting at page
FR	Le manuel commence à la page
IT	Istruzioni a partire da pagina
ES	Instrucciones desde la página
PT	Instruções da página
NL	Instructies vanaf pagina
SV	Instruktioner från sidan
DA	Instruktioner fra side
FI	Ohjeet sivulta
NO	Instruksjoner fra side
PL	Instrukcje ze strony
HU	Utasítások az oldalról
HR	Upute sa str
TR	Sayfadaki talimatlar

bachmaier® water:

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bachmaier® water:

Intended use: The bachmaier® water protects the ear from water and bacteria, keeps the ear canal dry and secure and prevents infections and inflammation in the inner ear. The customised design adapts perfectly to the ear canal and auricle and ensures a perfect fit without shifting or pressure. Thanks to the practical handle, it can be easily put in and taken out and ensures pure enjoyment in the water.

Special areas of application: water sports | swimming | kiting, surfing and sailing | splashing, showering and bathing | children with tympanostomy tubes

Instructions for correct use: The customised water protection is reusable. It is worn in the ear canal or auricle and protects the ears from impairments caused by incoming water when used correctly, i.e. when it is correctly fitted in the ear.

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If the water protection is worn with a connecting cord, the earmolds should not be removed from the ear by pulling on the cord, but by gripping the handle of the

earmold. Sudden or rapid removal of the earmolds from the ear canal can damage the eardrum.

Storage and care: Store your hearing protection in the supplied case at a maximum temperature of +55°C. Care for the hearing protection after each use by removing any ear wax residue with the wax stick provided and cleaning the earmolds with water, some mild soap or a damp cloth.

This product can be negatively affected by certain chemical substances. For further information, please contact the manufacturer.

Cleaning and Care with oto-fresh®: Discover our oto-fresh® products for optimal cleaning and care of your bachmaier® hearing protection! Available at your hearing care professional.

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IMPORTANT: The earmold marked 'R' fits in the right ear, the one marked 'L' in the left ear.

Notes on use: The water protection only provides its full protective effect if it fits correctly in the ear and is worn continuously in water. Have its fit and function checked regularly by a qualified dealer, as changes to the ear canal or damage can have a negative impact on the protective function. In this case, the earmould should be replaced. Ensure that the water protection is used correctly and checked in accordance with the manufacturer's instructions. If these recommendations are not followed, the protective effect may be impaired.

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bachmaier GmbH